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QUESTIONS AND ANSWERS

Q: How does the test work?

A: The FIRST RESPONSE™ Early Result Pregnancy Test detects hCG (human chorionic gonadotropin) your body makes during pregnancy. The hormone is found in urine when pregnant.

Q: When can I test with FIRST RESPONSE™ Early Result Pregnancy Test?

A: The FIRST RESPONSE™ Early Result Pregnancy Test is designed to detect hCG as early as 6 days before your missed period (5 Days before day of the expected period). In laboratory testing with early pregnancy urine samples, FIRST RESPONSE™ gave the following results.

NUMBER OF DAYS BEFORE THE EXPECTED PERIOD	% OF SAMPLES FROM PREGNANT WOMEN GIVING A POSITIVE RESULT
1	>99%
2	>99%
3	>99%
4	96%
5	76%

Q: How accurate is the test?

A: FIRST RESPONSE™ is >99% accurate from the day of the expected period in laboratory studies. However, some pregnant women may not have detectable amounts of pregnancy hormone in their urine on the day they test or may have miscalculated the first day of their period.

Q: What do I do if the result is positive (pregnant)?

A: Remember, this test is not intended to replace your healthcare professional's diagnosis. A positive (pregnant) test result shows that the pregnancy hormone (hCG) was detected. See your healthcare professional to confirm that you are pregnant.

Q: What do I do if the result is negative (not pregnant)?

A: You may not be pregnant, or it may be too early to tell. If you do not get your period within 7 days, you should retest with another FIRST RESPONSE™ Early Result Pregnancy Test. It is possible that you miscalculated the length of your cycle or your urine may not have had enough pregnancy hormone for the test to give a positive result. If upon retesting you get another negative result and your period has still not started, you should call your doctor.

Q: I have waited longer than 3 minutes, and still there are no lines in the Result Window. Is there anything wrong?

A: Your result should appear within 3 minutes. There should be at least one pink line in the Result Window at 3 minutes. The one pink line indicates that the test is complete and you have done it correctly. If there are no lines in the Result Window at 3 minutes, the test is invalid. You may not have enough urine on the Absorbent Tip. You may not have held the stick with the Absorbent Tip pointing down while putting the Overcap on after testing. You should retest with another device. Carefully follow all directions.

Q: How long will the result last?

A: The result can be read from 3 to 10 minutes. After 10 minutes, any change in the result should be disregarded.

Q: Do any drugs or medications affect the test?

A: Only medications containing the pregnancy hormone, human chorionic gonadotropin (hCG), can affect the result. Examples would be A.P.L., Pregnyl and Profasi. The test should not be affected by hormone therapies which have Clomiphene citrate (for example, Clomid and Serophene), alcohol, pain killers, antibiotics, or the contraceptive pill.

Q: Will the amount of liquid I drink affect the result?

A: Excessive fluid intake including water could dilute your urine, causing the amount of hCG present not to be detected properly. You should not drink too much liquid before testing.

To increase your chances for a healthy pregnancy for you and your baby, you should consider the following:

■ Maintain a well-balanced diet.

■ Stop smoking.

From conception through the first three months of pregnancy is the most important time. You need to see your healthcare professional immediately for proper care and nutrition counseling.

WARNING:

• According to the Surgeon General, women should not drink alcoholic beverages during pregnancy. Drinking alcohol creates the risk of birth defects.

• Do not disassemble device.

• Contains small parts.

• Do not eat any component of this device. If accidentally swallowed, seek medical attention.

72018492
FRIN-90127-29S

FIRST RESPONSE™ PREGNANCY

Early Result Pregnancy Test

First Response Early Result Pregnancy Test is designed to detect hCG (human chorionic gonadotropin). Test as early as 6 days before your missed period (5 days before the day of your expected period).

First Response is more than 99% accurate when used from the day of your expected period. Please read these instructions carefully before taking the test.

Questions? Please call us toll-free at 1-800-367-6022. M-F 7AM-5PM EST.

Visit us online at www.FirstResponse.com

INSTRUCTIONS

PLEASE READ THE INSERT BEFORE PERFORMING THE TEST.

• Do not use after the expiration date. Expiration date is stamped on the side of the carton. • Store in a dry place below 86°F (30°C). Do not freeze. • KEEP OUT OF THE REACH OF CHILDREN. • For in-vitro diagnostic use. Not to be taken internally. • Single-use device. Do not reuse.

1 Take the Test

- When you are ready to test, remove the test stick from the pink foil wrapper.
- Take off the pink Overcap and use immediately.
- Throughout testing, never hold the test stick with the Absorbent Tip pointing upwards.

Test In Your Urine Stream

- Hold the test stick by the Finger Grip. The printed side with Result Window must face away from your urine stream.
- Hold the Absorbent Tip in your urine stream for 5 seconds only.

OR

Test In Container

- To use the dip method, collect your urine in a clean, dry cup.
- Dip the entire Absorbent Tip in the urine for 5 seconds only.

- After testing, replace the Overcap, still holding the wet Absorbent Tip pointing down. Lay the stick on a flat non-absorbent surface with the Result Window facing up.

2 Wait 3 Minutes

- You may see a pink color moving across the Result Window.
- Time your test carefully. You will be able to read your results after 3 minutes.

3 Read Your Result

- At 3 Minutes, read your result in the Result Window.
- Important: read your result within 10 minutes.

PREGNANT

TWO PINK LINES in the Result Window.

One line may be lighter than the other.

NOT PREGNANT

ONE PINK LINE in the Result Window.

INVALID
NO LINES in the Result Window.

Test again with another First Response Early Result Test Stick

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